
EndoPilot Contra-Angle
User manual
Processing instruction

EN

ES

IT

PT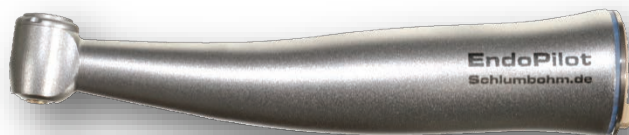

609 4110 v03 03.11.2021


Schlumbohm

Manufacturer information:

Schlumbohm GmbH & Co. KG
Klein Floyen 8-10
24616 Brokstedt
Germany

Telephone: +49 (0) 4324 - 8929 - 0
Telefax: +49 (0) 4324 - 8929 - 29
post@schlumbohm.de
www.schlumbohm.de

WEEE reg. no.: DE 88116129

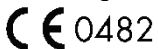The manufacturer reserves the right to change the information and data contained in this user manual without prior notice. On request, the user manual can be provided in different languages.

This user manual has been prepared with the greatest possible care. However, as errors can never be fully excluded, we would appreciate any information at any time so we can improve the documentation for you. Please contact us directly in such an event. Also, should you have any further questions, please do not hesitate to contact us.

(The data contained in this manual are referred to in the OEM document: 50669 -AEN Rev.003/06.10.2017)

Table of contents.....	Page
1. Introduction	3
2. Symbols used.....	4
3. First usage – Safety notes.....	5
4. Product description.....	5
5. Operation	5
5.1. Assembly and removal.....	5
5.2. Changing of the root canal instruments.....	6
5.3. Test run	6
6. Hygiene and maintenance	7
6.1. Pre-disinfection	7
6.2. Manual cleaning internal and external.....	7
6.3. Manual disinfection	8
6.4. Mechanical cleaning and disinfection internal and external	8
6.5. Lubrication.....	9
6.6. Testing after Lubrication	9
6.7. Sterilisation and storage.....	10
6.8. Approved sterilisation methods.....	10
7. Accessories.....	10
8. Technical data	11
9. Recycling and disposal.....	11
9.1. Recycling.....	11
9.2. Disposal of the contra-angle	11
9.3. Disposal of the packaging material	11
10. Liability	12
11. Service	12

1. Introduction

Customer satisfaction is our top priority. The present device was developed, manufactured and tested in line with all applicable legal and normative provisions.

For your safety and for the safety of your patients

Prior to first use, please read through the present instructions. The user manual is intended to give you information and explanations on the use of the device to ensure undisturbed, economic and safe treatments.

Intended use

Dental contra-angle for the mechanical preparation of root canals on the patient using root canal instruments.

Improper use may lead to damage of the contra-angle, resulting in and increased risk and hazard to the patient and third persons.

Professional use

This medical device is intended solely for use by professionals in dental medicine, in compliance with the applicable occupational health and safety regulations, safety precautions, and the present instructions for use. It may only be reprocessed and maintained by persons having undergone training on the prevention of infections, self-protection and protection of patients. Improper use, e.g. inadequate hygiene and improper maintenance, non-observance of the present instructions for use, or the use of

non-authorised accessories and spare parts, shall exempt us from any warranty or other claims.

2. Symbols used

Description of the symbols used.

Symbol	Description
 0482	The product complies with the requirements of EU Directive
	Warning notice: Consult the accompanying documents! Failure to comply with these instructions during operation may lead to damage of the device or injury of the user or patient.
	Caution! (if damage of objects might occur)
	Mechanical reprocessing in the thermal disinfectant
	Steam sterilisation
	General explanations, no risk for persons or objects
	Manufacturer's device type number
	Serial number of the device or of the component
	Medical Device
	Unique Device Identification (UDI), Data-Matrix-Code (GS1-Code)


3. First usage – Safety notes

Please make sure to provide for the correct operating conditions. Prior to use, please check the contra-angle for damages and loose parts (e.g. press button)

Hygiene and maintenance prior to first use the contra-angle was cleaned before delivery. Prior to first use, the contra-angle has to be maintained with oil and then sterilised. See “Maintenance with oil and sterilisation”(see chapter 6.5)

4. Product description

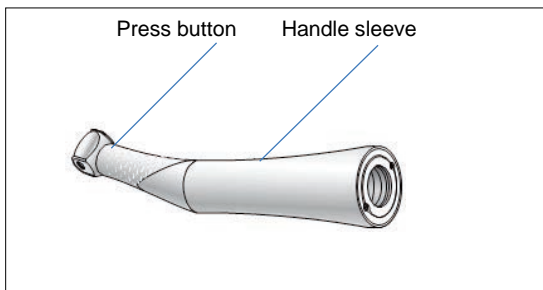
5. Operation

5.1. Assembly and removal

Do not insert or remove the contra-angle while in operation!

1. Connect the contra-angle to the motor until it is locked in place
2. Check if the contra-angle is tightly fastened to the motor
3. Remove the contra-angle by pulling it in axial direction

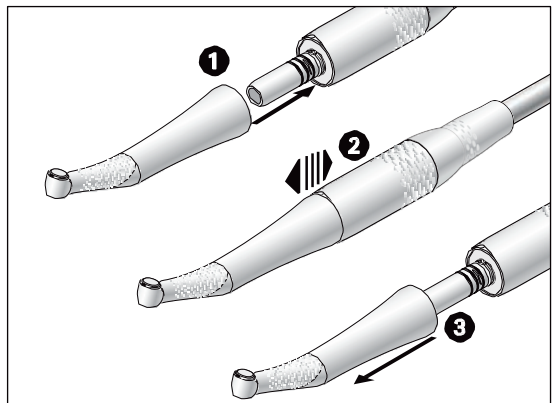
5.2. Changing of the root canal instruments

Root canal instruments

- Please make sure to solely use root canal instruments in perfect technical condition.
- Please observe the manufacturer's instructions.
- Please insert the root canal instrument only when the contra-angle is at stillstand.
- Don't ever touch the root canal instrument when it is still rotating or running out.
- Don't ever press the press button on the contra-angle while the contra-angle is still rotating or running out. This may result in the root canal instrument loosening, or the press button getting hot.
- Please make sure to always use a rubber dam.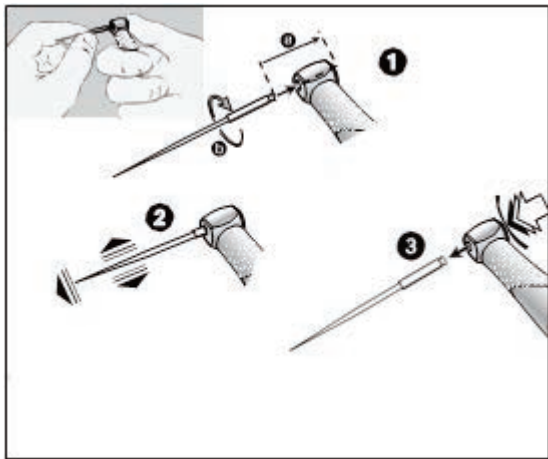

1. Insert the instrument until the stop position is reached (a) and turn it until it locks in place (b).
2. Check if the instrument is safely locked in place by pulling it in axial direction (b).
3. To remove the instrument, press the press button.

5.3. Test run

1. Insert the root canal instrument.
2. Start the contra-angle.
3. In case of functional disorder (for example vibrations, unusual noise, heating), please make sure to put the contra-angle out of operation immediately and contact the manufacturer (see chapter 11)

6. Hygiene and maintenance


- Please observe the specific guidelines, standards and provisions regarding cleaning, disinfection and sterilisation in your country.
- The contra-angle may be reprocessed either manually or mechanically.
- Please wear protective clothing.
- Remove the root canal instrument from the contra-angle.
- Disconnect the contra-angle from the motor.
- Please make sure to disinfect the contra-angle immediately after treatment to flush out any liquids that might have penetrated the contra-angle (for example blood, saliva, etc.), and to prevent clogging of the internal parts.
- After manual or mechanical cleaning and disinfection and maintenance with oil, the contra-angle needs to be sterilized.


6.1. Pre-disinfection

- Prior to cleaning, remove stubborn contamination with disinfecting wipes.
- Please make sure to only use disinfectants which do not have a protein fixing effect.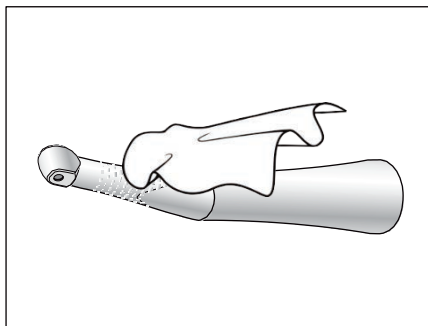

6.2. Manual cleaning internal and external

1. Rinse contra-angle with demineralised water (<math><38^{\circ}\text{C}</math>) using a brush
2. Remove any residual liquids with an absorbent cloth and dry with compressed air.

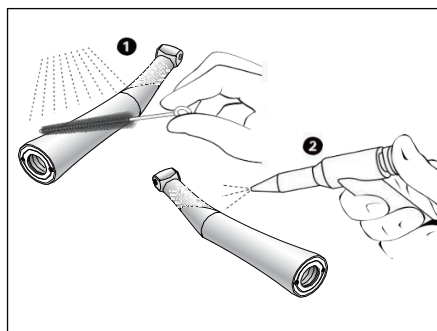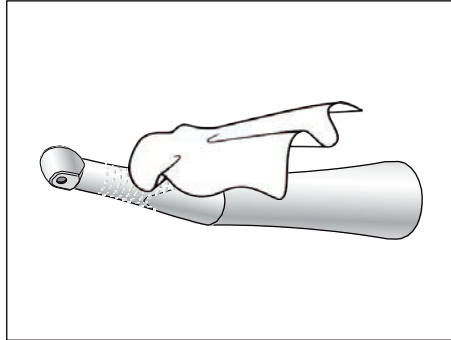
The contra-angle must not be immersed in a disinfecting solution or ultrasonic bath!

6.3. Manual disinfection

We recommend disinfecting the contra-angle with disinfecting wipes

-  Make sure to only use certified disinfecting agents free from chlorine from officially recognised institutes.
- Please follow the instructions on proper disinfection of each manufacturer.

After manual cleaning, disinfection and maintenance with oil, final thermal disinfection of the contra-angle (without packaging) or sterilization (in a suitable packaging) in a steam sterilizer of class B or S (in compliance with EN 13060) is necessary.

6.4. Mechanical cleaning and disinfection internal and external

The contra-angle is labelled with this symbol, it can therefore be cleaned and disinfected in a thermal disinfecter.


-  We recommend preparing the contra-angle in a thermal disinfecter. Please follow the instructions of the manufacturers of each device or cleaning and disinfecting agent.
-  After thermal disinfection, please ensure that the all internal and external parts of the contra-angle are completely dry. Any residual liquids have to be removed with compressed air.
- Immediately after thermal disinfection, the dry contra-angle must be lubricated with oil.

6.5. Lubrication

1. Lubricate the contra-angle with oil, for example with W&H Service Oil F1, M400. Please follow the instructions of use printed on the aerosol can.


or

2. Lubricate the contra-angle in an automatic instrument maintenance device, for example in the W&H Assistina (see instructions for use coming with the device). Recommended maintenance cycles: Imperative after each internal cleaning, before each sterilization, or after 30 minutes of use, or once a day. After maintenance, please place the contra-angle in an upright position on a clean disposable cloth. This way, excessive oil may drip off so that it cannot get into the motor.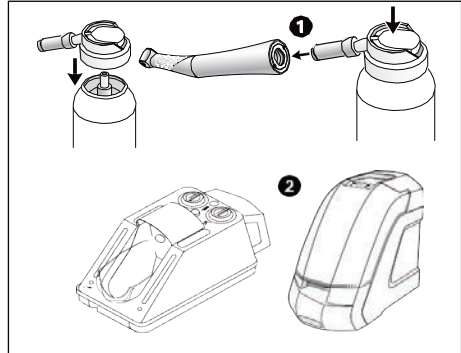

6.6. Testing after Lubrication

1. Hold the contra-angle with the head pointing downwards.
2. Let the contra-angle run briefly to remove excessive oil. To this end, start the calibration function on the EndoPilot. In case of dirt leaking from the contra-angle, the entire hygiene and maintenance procedure must be repeated.
3. Wipe the contra-angle with absorbent paper or a soft cloth.

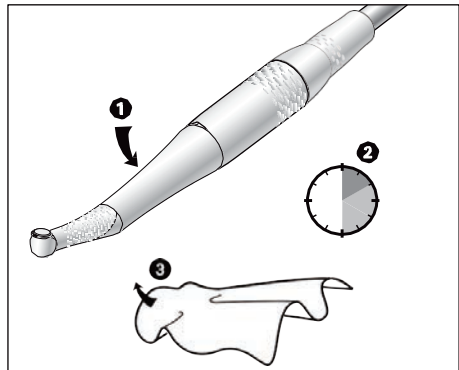6.7. Sterilisation and storage

Sterilisation should be carried out in compliance with EN 13060, class B

- Please observe the instructions of the device manufacturer.
- Prior to sterilisation, the contra-angle has to be cleaned, disinfected and lubricated with oil.
- For sterilisation, please make sure to seal the contra-angle in a suitable sterilisation packaging for sterile articles in compliance with EN 868-5.
- Please make sure that the device to be sterilised is absolutely dry.
- Store the sterile device in a dust-free and dry place.

6.8. Approved sterilisation methods

Please observe all specific guidelines, standards and provisions in your country

- Steam sterilisation class B with sterilisers as per EN 13060.
Minimum sterilisation holding time 3 minutes at 134 °C or
- Steam sterilisation class S with sterilisers as per EN 13060
Must be expressly authorised by the manufacturer of the sterilizer for sterilisation of contra-angles.
Minimum sterilisation holding time 3 minutes at 134 °C

7. Accessories

The equipment used must be suitable for dental instruments and contra-angles.

For example: W&H Service Oil F1, MD-400 and ISO-E spray nozzle with spray adapter for handpieces and contra-angles

8. Technical data

Gear ratio	1:1
Motor coupling in compliance with standard	ISO 3964
Recommended root canal instruments* Shank diameter of the instruments (Type 1) acc. to ENISO 1797-1:1995	NiTi files for the mechanical preparation of root canals Ø 2,35* mm
Minimum insertion depth	until the instrument is firmly locked in place
A maximum motor rotational speed for NiTi files of 2000 rpm: equals a rotational speed of operation of 2000 rpm	2000 rpm

* By choosing the right conditions of operation, the user has to make sure to exclude any hazard to the user, patient or third persons. Please observe the instructions of the root canal instrument manufacturer (in terms of speed, torque and type of movement)

9. Recycling and disposal

9.1. Recycling

We feel particularly committed to protect the environment. Our contra-angle and the packaging were designed as environmentally compatible as possible.

9.2. Disposal of the contra-angle

Please observe the specific laws, guidelines and provisions in your country.

9.3. Disposal of the packaging material

The packaging materials were duly selected to allow an environmentally friendly disposal, therefore, all components can be recycled. Please dispose of all no longer required packaging materials in the collection and recycling system. In so doing, you can contribute to recycle raw materials and to reduce waste.

10. Liability

Schlumbohm® warrants this product against defects in materials and workmanship for the period of one year from the date of the original invoice. The product warrant provided by Schlumbohm® includes the repair or the replacement of the entire device or individual parts. The decision whether to replace or repair is entirely up to the manufacturer.

In the event of an alleged defect during warranty, the customer has to inform the customer service immediately. The customer service will give further instructions. Normally you will be asked to return the contra-angle. The costs of returning the product are at the sender's expense.

Application errors exclude a warranty. Schlumbohm® does not warrant for wear and contamination of the handpieces and contra-angles.

Schlumbohm® declines all responsibility for any damage caused as a result of the clinical application of its products. Irrespective of whether or not such use is associated with other medical devices (e.g. pacemakers).

11. Service

In case of functional disorder, you are requested to immediately contact your dealer or the manufacturer. Repair and maintenance may only be carried out by an authorised service partner.

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